In-Office WHITE SENSE BLEACHING ULTRA HP

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In-Office WHITE SENSE BLEACHING ULTRA HP

- 3 syringes containing 1 ml of White-FIX desensitizing and fixative mousse
- 7 needles + 3 connectors
- 1 bilingual instruction manual for the dentist
- 1 instruction manual for the patient

Application:

- Immediate whitening of living and non-living teeth in the office that have changed yellow, gray, brown, etc.
- This kit can be used for effective whitening of the entire teeth, parts of the upper and lower teeth, or selected points on some teeth.

Important warning:

- Be sure to use gloves and safety glasses while working. To protect the patient, if possible, use safety glasses for the patient and lubricate the patient's mouth with a little Vaseline to protect the gel and skin necrosis.
- The duration of working time and storage of bleaching gel after mixing the two components is maximum 5 minutes. Immediately after mixing the two components, the gel should be placed on the patient's tooth and consumed, and avoid keeping the gel longer than this time. Before mixing the two components, it is better to apply the gingival barrier gel and light cure it, and then prepare the bleaching gel.
- Avoid storing the prepared and mixed gel. In this case, its efficiency is greatly reduced.
- It is necessary for the respected dentist to present the patient guide sheet, which is in the White Sense kit box, to the patient.
- The bleaching process may aggravate tooth sensitivity. Therefore, after finishing the work, in order to increase the durability of the whiteness of the teeth and reduce the possible sensitivity, it is necessary to use the White-FIX desensitizing and fixator mousse, according to the instructions mentioned in the office.
- The use of this product is not recommended for pregnant, lactating women and people under 15 years of age.
- This product is not suitable for whitening restorative materials such as composite and laminate, etc. and has no effect on it.
- People who are allergic to Hydrogen Peroxide and its derivatives should avoid using this product.
- This product can only be used for people who have healthy gum tissue and healthy teeth without decay.

- Bleaching gel, Dora-Dam gingival barrier gel and White-FIX fixator mousse or its residue (even a small amount) should not be swallowed in any way, so precautions should be taken by a dentist.
- Avoid direct contact of bleaching gel with gums, saliva, palate, roof of the mouth, lips, face and soft tissue.
- If Hydrogen Peroxide comes into contact

with the gums or other tissues, the process of whitening and burning of the tissue occurs, which may last up to 1 hour and then disappear. You can use a moisturizing cream to reduce the burning sensation.

■ The clinical results of WhiteSense show that the most whitening and durability after bleaching is in patients who use the 23% Carbamide Peroxide Home bleaching kit or the WhiteSense whitening Pen for 5 weeks after performing office bleaching with a 40% kit in the office.

First aid:

- In case of contact with skin/tissues: wash immediately with plenty of water, use moisturizing cream to reduce irritation. If the symptoms of burning, white tissue, inflammation, redness and heat persist for more than 1 hour, go to the medical center immediately.
- In case of contact with the eyes: open the eyes wide and rinse with running water for 15 minutes and immediately go to the medical center.
- If swallowed: rinse mouth with plenty of water. Then wash with a solution of water and baking soda (1 spoon in a glass). Drink plenty of milk and water and go to a medical center immediately.


Storage and maintenance:

- Keep the product away from heat and light. Avoid freezing the product and putting it in the freezer. It is better to store the product in the refrigerator. Do not use after the expiration date.

Possible side effects:

Tooth sensitivity to cold and heat, mild tooth pain, these symptoms are moderate and mild, up to 5 days after using the product. In case of unusual complications and side effects other than those mentioned, contact the emergency number: +(98)9058384458.

Manufacturer:
Dorsan Darman Co.
www.dorsandarman.com

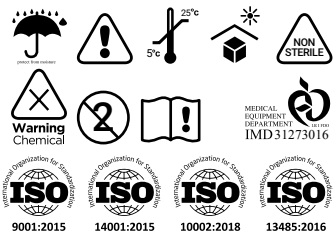- In order to increase the effectiveness of the bleaching process, for the bleaching of each patient, the process of making the gel and placing the gel on the teeth is done twice. For 2 times and 30 minutes each time (60 minutes in total)
- For each patient, 2 Activator syringes, 2 plastic capsules containing liquid, 2 empty syringes for filling Hydrogen Peroxide, 1 fixative syringe are used.

Introductions:

1. First, check the patient's mouth and make sure that there is no pest, inflammation, redness, wound or gum disease. It is better to clean the sick tooth with Pumice powder.

2. Determine the current color of the patient's teeth and the patient's desired color after bleaching using the VITA shade.

3. Place the retractor in the patient's mouth and apply the Vaseline cream in the kit to the part of the lip that may come in contact with the bleaching material.

4. Dry the teeth and gums completely so that saliva does not interfere with the work. Then put the Dora-Dam gingival barrier gel using a black needle on the top and bottom of the gum in sufficient amount and make sure that the entire gum area is covered with the gel.


5. Light cure it for 15 to 20 seconds to harden the

gingival barrier gel. If you apply a thick layer, do the light curing process twice.


6. After that, take a plastic capsule containing liquid Hydrogen Peroxide and an empty 1 ml syringe and carefully insert the needle of the empty syringe into the plastic capsule and slowly inject 0.8 ml equivalent to 80 syringe units (no more, no less). Fill the liquid into the syringe. Remove the needle head and connect a connector to the syringe containing liquid.


For convenience, more solution is in the plastic capsule. But only the amount of 80 liquid units should be fill.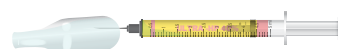

7. Remove the protective cap of the syringe containing the activator (0.2 ml of cream) and connect the syringe to the connector connected to the syringe containing Hydrogen Peroxide liquid. Make sure the two syringes are completely and firmly connected by the connector.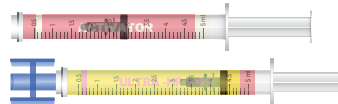

8. Mix the Hydrogen Peroxide and the Activator well by pressing the piston of the syringe back and forth. Do at least 10-15 times to form a gel.

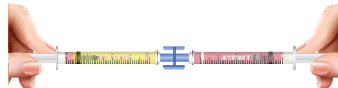9. Keep the final gel in a syringe and remove the connector and the other syringe and connect the needle head. It is better not to lift the needle head of gel on a pad to prevent splashing and control the gel flow.

10. Using a needle, place a thin layer of gel on the teeth. It is better not to lift the needle head when moving from one tooth to another and use a continuous movement.

11. Keep the gel on the teeth for 30 minutes. Then carefully suck out all the gels. Check the tooth color with Shade Vita and repeat steps 5-10 once more. When the patient reports pain, stop the bleaching process and follow the steps of using the White-FIX fixator mousse.

12. After reaching the desired whiteness, remove the gel from the teeth with suction and wash the teeth with a gentle stream of water and dry them. Remove the gingival barrier gel from the gum by lifting from one end.

13. While the retractor is in the patient's mouth, connect the needle head to the White-FIX mousse fixator syringe and carefully coat the entire surface of the tooth with this mousse. Remove and remove the excess amount of mousse on the gums with an ear cleaner. In this case, the patient can go through these steps in the waiting room. It is recommended to leave this mousse on the teeth for 20-30 minutes for better effectiveness, stability of whitening, increasing the strength of the teeth and removing possible sensitivity. If the respected dentist or the patient does not have enough time for this step in the office, this mousse can be provided to the patient to be placed on the surface of the teeth at home and then rinse the teeth with water.

14. After that, suck the mice from the teeth. Remove the retractor from the patient's mouth and the patient can brush his mouth and teeth.

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